

BenaCel®

Hemostatic Dental Dressing

APPLICATIONS

- ❖ Manages surgical bleeding
- ❖ Seals sinus perforation
- ❖ Reduces dry socket formation

BenaCel® Dental Dressing is made of biocompatible oxidized cellulose and contains no chemical additives.

When placed in the extraction socket, BenaCel® Dental Dressing forms a gelatinous scaffold in the blood extrudate, facilitating the development of a stable blood clot to stop bleeding and prevent dry socket formation. BenaCel® Dental Dressing adheres to moist oral mucosa and forms a barrier, protecting the wound from further irritation and pain. The material dissolves in a few days and is safe if swallowed.

BenaCel® may also be used as a wound dressing for the temporary management of oral surgical wounds, such as operative, postoperative, donor sites, and traumatic injuries. Made of plant based material, BenaCel® is non-allergenic and non-immunogenic. Several sizes are available for various wound configurations.



AVAILABLE SIZES

- ❖ 5 x 7mm plugs - for anterior conical extraction sites
- ❖ 6 x 8mm plugs - for posterior conical extraction sites and sinus perforations
- ❖ 15 x 15mm sheets - for mandibular molars and buccal roots of maxillary molars
- ❖ 50 x 50mm sheets - for surgical wounds, such as operative, postoperative, donor sites, and traumatic injuries

BenaCel®

Dental Dressing

POST-EXTRACTION WOUND MANAGEMENT

Fresh 3rd molar extraction site



Remove granulation tissue



Obtain BenaCel® from blister pack



Insert BenaCel® into wound site



With BenaCel® dental dressing, management of post-extraction wound healing becomes as easy as 1, 2, and 3. (1) After extraction, remove granulation tissue, (2) pack 1 or 2 pieces of BenaCel® dental dressing into the wound, and then (3) have the patient bite down on a sterile gauze pack until bleeding stops. The gelatinous material helps to form a stable blood clot, preventing dry socket formation. BenaCel® dissolves within days and is safe if swallowed.

CLINICALLY PROVEN RESULTS

Maxilla 2nd Molar



BenaCel® Placed



1st Post Op

Mandibular 3rd Molar



BenaCel® Placed



1st Post Op

Maxilla Central Incisors



BenaCel® Placed



1st Post Op